

## Laboratory Genetic Metabolic Diseases

Amsterdam UMC, location AMC  
Lab. Genetic Metabolic Diseases (F0-132)  
Meibergdreef 9  
1105 AZ Amsterdam, The Netherlands

www.labgmd.nl  
gmz\_metab@amc.nl  
Tel: +31 (0)20 - 566 5393  
Fax: +31 (0)20 - 696 2596

*Dr. F.M. Vaz - clinical biochemist IEM*  
*Dr. S.M.I. Goorden - clinical biochemist IEM*

*Dr. W. Kulik - clinical laboratory geneticist*  
*Dr. M.M.C. Wamelink - clinical laboratory geneticist*

### Protocol for collection and pretreatment of blood samples for the determination of oxalate in plasma

- Collect at least 2 ml of heparin blood in an ice-cooled (0° C) tube and centrifuge for 10 minutes at 1700 x g at 4° C within 1.5 hrs (also keep on ice before centrifugation).
- Transfer 200 µl of plasma in a 4 ml glass screw capped vial, in quadruplicate. (Do not use plastic tubes, this causes interference with the analysis!)
- Add 30 µl of concentrated HCl (37%) to each vial, mix immediately using a vortex and close the vials securely.
- Store the vials at -20° C until shipment.
- Send the frozen samples on sufficient dry ice to our laboratory by (international) courier together with a completed test request form Metabolite Diagnostics (available at [www.labgmd.nl](http://www.labgmd.nl)).